




InStove

Autoclave Adapter Ring Instructions

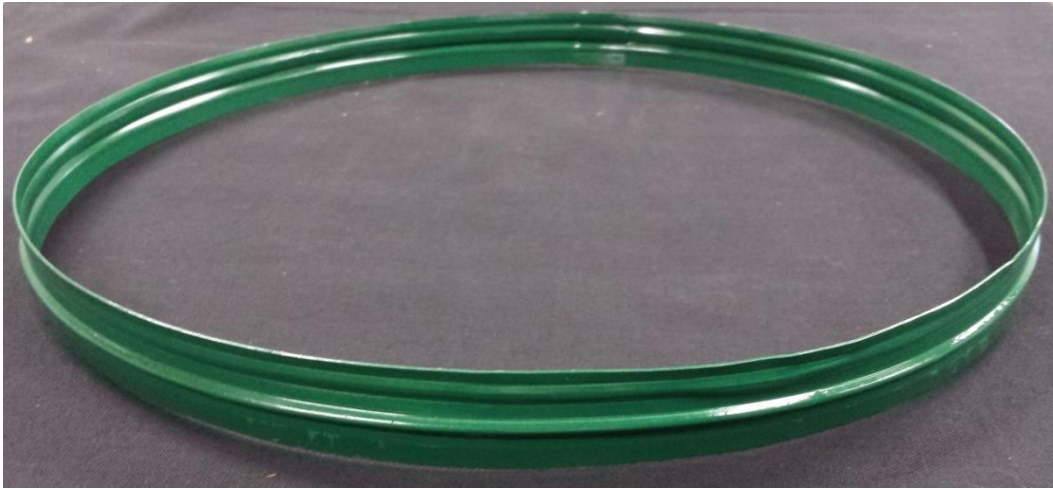

Adapter Ring



Adapter Ring Installed in a 60 Liter Stove

Installation and Use: Insert the adapter ring into the stove when the autoclave is to be used, as shown above. Place the autoclave into the stove with adapter ring installed. Operate the stove as you would normally.

Notes:

- Be sure to place the narrower end of the ring into the stove, first, so that it fits down inside the rim of the stove.
- **DO NOT** install or remove the adapter ring while the stove is lit or still hot
- Remove the adapter ring before using the stove with a cooking pot or other accessories.